mesoestetic



mesohyal DMAE ELASTICITY AND FIRMNESS.

mesohyal DMAE is a reaffirming solution combining dimethylaminoethanol (DMAE) with a base of hyaluronic acid.

DMAE is a precursor of acetylcholine, a neurotransmitter involved in muscle tone increase. Its action returns firmness to the tissues undergoing the aging process, effectively counteracting flaccidity.

Based on the diagnosis of the patient's aesthetic disorder, **mesohyal DMAE** is the ideal complement to prepare cocktails with other products from the **mesohyal**⁹ range:

- · mesohyal HYALURONIC: with skin moisturising and rejuvenating effect.
- · mesohyal NCTC 109: with intensive biorevitalising effect.
- · mesohyal VITAMIN C: with antioxidant effect.
- · mesohval BIOTIN: to reactivate cell metabolism.
- · mesohyal CARNITINE: to increase energy output.
- · mesohyal ARTICHOKE: to activate lipolytic processes.
- · mesohyal MELILOT: for cellulite with vascular component.
- · mesohyal ORGANIC SILICON: to regenerate and restructure the skin tissue.

For cocktails with **mesohyal HYALURONIC**, load this first because of its high viscosity. A homogeneous mixture will thus be obtained.

The **TREATMENT** is applied over 20 to 30 minutes, and the patient can return immediately to their activities of daily living. The therapeutic protocol will be based on the patient's age and skin condition, requiring 5 to 10 sessions at 1 or 2 week intervals.

Although the repairing and antiaging action of **mesohyal DMAE** is immediate and long-lasting, it does not stop the effects of bad habits regarding figure, or aging, so regular maintenance treatments will be required every 6/8 months to preserve the skin's healthy appearance. Based on the result obtained, this period may be shortened or prolonged.



mesoestetic



Specifications:

Composition:	DMAE / Deanol Bitartrate (3%) Sodium hyaluronate: Biotechnological origin / non cross-linked Concentration: 2,5 mg/ml
Indications of use:	Face and body skin flaccidity
Treatment areas:	Especially indicated for the face, neck, neckline and back of the hands, as well as the arms, back, chest, abdomen, periumbilical area, flanks, and thighs.
Presentation:	Class I 5 ml ampoule (monosilicate glass) Package containing 20 ampoules
Storage conditions:	Keep under 25°C Do not expose to high or low temperatures Avoid direct exposure to light
Injection techniques:	Sterile syringes and needles for mesotherapy techniques / manual or multipuncture gun
Injection depth:	Superficial dermis

mesoestetic

