

mesoestetic®



mesohyalTM ORGANIC SILICON

SKIN TISSUE REGENERATION AND RESTRUCTURING.

mesohyal ORGANIC SILICON is a regenerating, restructuring solution combining organic silicon with a hyaluronic acid base.

Organic silicon acts for the purpose of stabilising and maintaining skin tissue structure, stimulating fibroblasts and enhancing the biosynthesis of collagen fibres, elastin, and proteoglycans.

The moisturising and microfilling action of the hyaluronic acid contained in **mesohyal ORGANIC SILICON** increases skin elasticity, improving skin moisturising retention and reducing the signs of aging.

Based on the diagnosis of the patient's aesthetic disorder, **mesohyal ORGANIC SILICON** is the ideal complement to prepare cocktails with other products from the **mesohyal**® range:

- **mesohyal HYALURONIC**: with skin moisturising and rejuvenating effect.
- **mesohyal NCTC 109**: with intensive biorevitalising effect.
- **mesohyal VITAMIN C**: with antioxidant effect.
- **mesohyal DMAE**: with firming effect.
- **mesohyal BIOTIN**: to reactivate cell metabolism.
- **mesohyal CARNITINE**: to increase energy output.
- **mesohyal ARTICHOKE**: to activate lipolytic processes.
- **mesohyal MELILOT**: for cellulite with vascular component.

For cocktails with **mesohyal HYALURONIC**, load this first because of its high viscosity. A homogeneous mixture will thus be obtained.

The **TREATMENT** is applied over 20 to 30 minutes, and the patient can return immediately to their activities of daily living. The therapeutic protocol will be based on the patient's age and skin condition, requiring 5 to 10 sessions at 1 or 2 week intervals.

Although the repairing and antiaging action of **mesohyal ORGANIC SILICON** is immediate and long-lasting, it does not stop the effects of bad habits regarding figure, or aging, so regular maintenance treatments will be required every 6/8 months to preserve the skin's healthy appearance. Based on the result obtained, this period may be shortened or prolonged.



mesohyal ORGANIC SILICON is supplied in 5 ml ampoules. Each ampoule is for single use and for a single patient.

Specifications:

Composition:	Silicio 2,7 mg/5ml Sodium hyaluronate: · Biotechnological origin / <i>non cross-linked</i> · Concentration: 2,5 mg/ml
Indications of use:	Prevention of aging Face and body skin flaccidity Stretch marks in primary stage Scars
Treatment areas:	Especially indicated for the face, neck, neckline and back of the hands, as well as the arms, back, chest, abdomen, periumbilical area, flanks, and thighs.
Presentation:	Class I 5 ml ampoule (monosilicate glass) Package containing 20 ampoules
Storage conditions:	Keep under 25°C Do not expose to high or low temperatures Avoid direct exposure to light
Injection techniques:	Sterile syringes and needles for mesotherapy techniques / manual or multipuncture gun
Injection depth:	Superficial dermis